

HEARTWARE EXPLANT

1. SET FLOW ALARM LIMIT TO 1.0 L/MIN UNDER ALARMS TAB
2. SET HEMATOCRIT TO 20% UNDER PATIENT TAB
3. “NEUTRAL” OR “ZERO” FLOW IS 1800 RPMS WHEN YOU GO ON BYPASS
4. TURN OFF VAD USING **RED STOP** BUTTON ON SCREEN
5. WHEN YOU ARE READY TO DISCONNECT VAD, REMOVE DATA CABLE AND PUT ON **RED ALARM SILENCER** ON DATA PORT
6. REMOVE DRIVELINE THEN 2 POWER SOURCES



HEARTMATE 2 AND 3

EXPLANT

1. CONNECT TO POWER MODULE AND CLINICAL SCREEN
2. REMOVE BACKUP BATTERY FROM POCKET CONTROLLER WITH GREEN SCREWDRIVER
3. **ONLY FOR HM3** YOU MUST TURN PUMP TO 3000 RPMS FOR A MOMENT TO TRIGGER LOW FLOW ALARM, SILENCE THE ALARM ON CLINICAL SCREEN, THEN GO TO ALARM TAB AND HIT “EXTENDED SILENCE” **WARN ANESTHESIA** BEFORE YOU DO THIS, THEN GO BACK UP TO NORMAL SPEED
4. “NEUTRAL” OR “ZERO” FLOW WITH HM2 IS 6000 RPMS, HM3 IS 3000 RPMS WHEN YOU GO ON BYPASS
5. TURN OFF BOTH PUMPS BY PULLING THE CABLE WITH THE **ORANGE** DOT OUT OF THE POWER MODULE

